

EXPORTING SCANS FROM ITERO

NEO
LAB


STEP 1

In a web browser, go to [MyAligntech](#) to access your iTero account




Email:

Password:



STEP 2

Find the correct case and click on the [hyperlink](#) code next to the patient's name

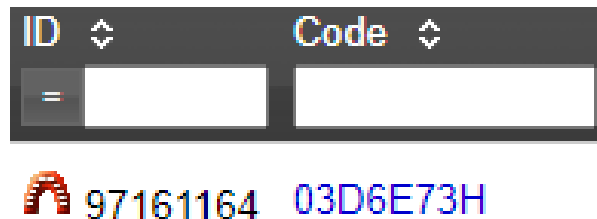

STEP 3

On the Order Information page, select the [hyperlink](#) labeled "Export (OrthoCAD)"

[Open File](#) [Export \(OrthoCAD 3.5 or higher\)](#) [Change Order](#)

**If the scans don't appear, you may need to update OrthoCAD before continuing*

For any issues accessing OrthoCAD, please consult this [guide](#)

STEP 4

In OrthoCAD Export, select the following settings:

| | |
|--------------|--|
| Export Type: | Open Shell |
| Data Format: | File per Arch (arches oriented in occlusion) |
| File Format: | STL |
| File Size: | |
| Folder Name: | PATIENTNAME |

STEP 5

Click "Export" and on the new pop up window select "Yes" to view the scans

Successfully finished.
Model data was exported to C:\Cadent\Export\PATIENTNAME\
Would you like to open the exported file's location?

Yes

No

STEP 6

In the opened folder, select the file(s) and drag them over to the RX Wizard prescription



97161164_shell_o
cclusion_l



97161164_shell_o
cclusion_u



Drop here or click to upload

97161164_shell_occlusion_u.stl

97161164_shell_occlusion_l.stl